


**Typenex.**

## **TypeSafe™** **Segment Sampling Device**

**EASY – SAFE – HYGIENIC**

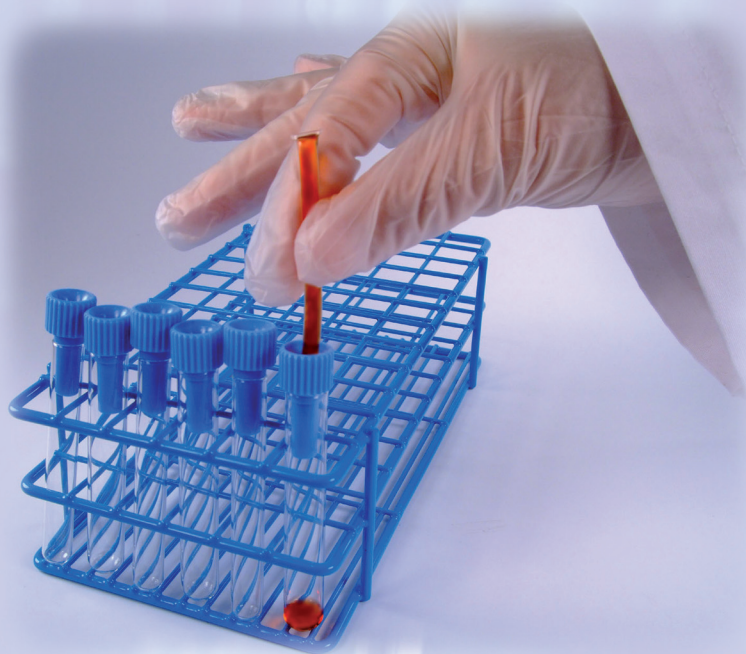

- TypeSafe™ allows for segment puncture without scissors for cross-matching procedures in the lab
- The large canula allows cells to pass through with ease
- Top and bottom of device are easily identified by technicians in the blood bank
- Ribbed exterior allows for easy handling with gloved hands
- Fits into 10 ml and 12 ml tube sizes
- Wide mouth fits most standard segment sizes




STRADIS-MED GmbH  
Woogstrasse 48  
D-63329 Egelsbach  
Amtsgericht Offenbach  
HRB 42891  
Office: +49 6103 - 70 64 22  
Fax: +49 6103 - 70 64 229  
info@stradis-med.de  
www.stradis-med.de



## Why Use The TypeSafe™ Segment Sampling Device?

The TypeSafe™ Segment Sampling Device is designed for single, in-vitro, diagnostic use, so technicians do not have to use scissors for cross-matching procedures in the lab.

- Insert the narrow end of the TypeSafe™ segment sampling device into the test tube
- Make sure the tube is properly aligned in the holder
- Always wear proper safety attire when working with blood. Insert blood segment into TypeSafe™ and gently press until the needle has pierced the segment
- Apply pressure to the segment, forcing blood into the test tube
- Do not squeeze until needle has pierced segment
- Be sure to place the TypeSafe™ directly on top of test tube, so the narrow end is facing straight down, reducing angles and gaps
- When desired sample has been dispersed into the test tube, remove the blood segment and the TypeSafe™ device. Properly dispose of both in designated hazardous medical waste receptacles
- For single use only
- For in-vitro diagnostic purposes only

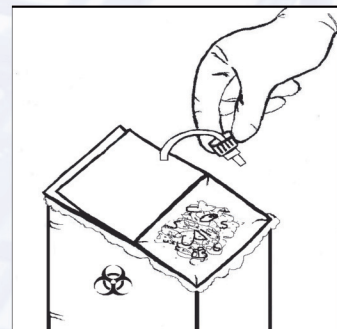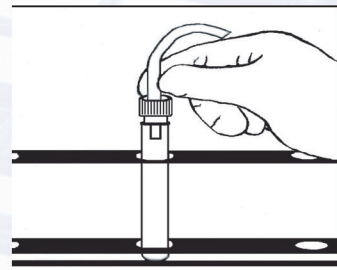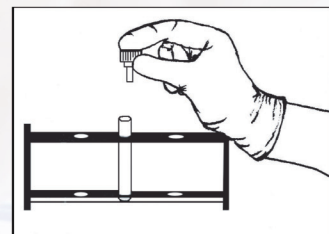

### Lab Supply Basics:

- **Highlights:** FDA Cleared
- **Indications:** Single, In-vitro diagnostic use
- **Packaging:** 500 units per bag,  
2 bags per case (1,000 units/case)
- **Catalog Codes:** 4R4750



QualRep Services B.V.  
P.O. Box 302  
1440 AH, Purmerend  
The Netherlands  
T: 0031 (0) 85 7440000  
F: 0031 (0) 299 660271

STRADIS-MED GmbH  
Woogstrasse 48  
D-63329 Egelsbach  
Amtsgericht Offenbach  
HRB 42891  
Office: +49 6103 - 70 64 22  
Fax: +49 6103 - 70 64 229  
info@stradis-med.de  
www.stradis-med.de